



QUICK START GUIDE

Right out of the box, all 8 curing modes can be accessed on the menu by repeatedly depressing the “MODE” button closest to the charging end of the light, in succession:

Normal	10 seconds	1200 mw/cm ²
Normal	20 seconds	1200 mw/cm ²
Soft Start	15 seconds	1400 mw/cm ²
Pulse	5 seconds	1600 mw/cm ²
Pulse	10 seconds	1600 mw/cm ²
Turbo	3 seconds	1600 mw/cm ²
Turbo	5 seconds	1600 mw/cm ²
Turbo	10 seconds	1600 mw/cm ²

With the same “MODE” button you may customize the light so that only the curing modes that will be used daily will display on the menu:

1. Press and hold the MODE button
2. Wait until “Selection mode” appears and stops flashing on the LED screen.
3. Use the MODE button to scroll through each available curing mode.
4. In selection mode, you can turn on or turn off each mode by depressing the “Transillumination Button (light-bulb symbol)” in succession. A small circle next to the curing mode description indicates that the mode will not show on the menu after programming. The larger sunburst symbol indicates that the mode will be available on the menu after programming.
5. When finished programming, simply press and hold the MODE button again until you hear a beep. Now using the mode button will scroll through only the modes that have been selected by the user.

The power button next to the light output is also the light trigger.



NATIONAL DENTAL INNOVATIONS